

## 【Ultrathin Bunion Corrector】

### Intended use and characteristics

Because of the fixation around the heel the bunion corrector constantly applies pressure and does not slip. The fabric is only 0.4mm thin which makes it comfortable to wear underneath socks and in shoes. It is not noticeable and comfortable to wear even in the business shoes. A hole in the tip of the big toe relieves the supporter's pressure to ensure wearing comfort.

### How to wear

Ultrathin Bunion Corrector is anatomically shaped for either the right or left foot. Confirm on the size tag for which foot this product can be used. This product is intended to be worn directly on the skin.



1

Open up the product and put the tips of the toes through the opening.



2

Make sure to put the big toe in the part with the little hole in the tip. Pull so that the back of the foot fits through the big opening.



3

Pull up the heel strap and place it on the heel.



4

Adjust the main part on the back of the foot by pulling it in direction of the big toe. Also moderately twist the material around the big toe so that the hole comes to the middle of the tip of the big toe.

### Washing Instructions

- Handwash below 30 degrees.
- Do not bleach. Use a neutral detergent which does not contain whitening or bleaching ingredients.
- Dry the product in a place with good ventilation away from direct sunlight, i.e. in the shadow or inside a room. Do not tumble dry.
- Do not iron.
- Do not dry clean.

### Caution

- ⚠ Do not use this product other than explained in this instruction manual as this can lead to damages or injuries.
- ⚠ This product contains chemical fibers. In case the product causes skin irritations stop wearing the product immediately and consult a doctor.
- ⚠ Make sure the fastener does not stick to other clothes as this can cause damage of the clothes.

MDSS GmbH  
Schiffgraben 41, 30175 Hannover, Germany

Daiya Industry Co., Ltd

For questions or inquiries regarding this product please contact

Daiya Industry Co., Ltd.  
1253 Ofuku, Minamiku, Okayama City, 701-0204, Japan  
TEL: +81-86-282-1245  
Email: info@daiyak.co.jp  
Website: <http://www.daiyaindustry.com/>

### Storage Instructions

- ☀ Do not expose the product to direct sunlight, high temperatures or other conditions that might cause damage.
- ☔ Store in a dry place with good ventilation.

### Disposal Instructions

Proper disposal differs in each country and might even vary within the states, communities etc. of one country. To ensure the proper disposal of this product please confirm the way of disposal with your local guidelines for textile product disposal.

### Material

Nylon, Polyurethane

### Size

Size in Centimetres

Size	Foot length	Big toe circumference	Instep circumference
XL	27 - 29cm	8.5 - 9.5cm	26 - 28.5cm
L	25 - 27cm	8 - 9cm	24 - 26.5cm
M	23 - 25cm	7.5 - 8.5cm	22 - 24.5cm
S	21 - 23cm	7 - 8cm	20 - 22.5cm
XS	19 - 21cm	6.5 - 7.5cm	18 - 20.5cm

LOT see package imprint

Latest revision of this instruction manual:  
September 2014