

# Instructions for use of Cinderella corrective sleeves



2 in 1 foot corrective sleeve. Reliefs and realigns a **mild to moderate** level Hallux Valgus (Bunion) and Digitus Quintus Varus (Bunionette, Tailor`s bunion)

## Product description

The **Cinderella** corrective sleeve realigns the big toe and the little toe to their more natural positions. Please note, that a mild to moderate level of both bunions can be treated with this product. In case of high level Hallux Valgus and / or Digitus Quintus Varus, the corrective sleeve may relieve your bunions, a realignment however may not be possible.

**Cinderella** spreads both, the big and the little toe and is opening all other toes accordingly. **Cinderella** heightens the longitudinal arches and tightens the ligaments around the metatarsal joints. In addition **Cinderella** prevents to several impacts to the foot as they are hammer toes, claw toes and clonus.

## Application period

Give your feet time to get accustomed to the product. In case of pain to your toes or the whole foot, please do not exceed a daily wearing time of max. 2 hours. As soon as your feet have gotten accustomed with the corrective sleeve, you can wear it up to 12 – 24 hours. In the event of long wearing periods, please take care to your foot hygiene factors.

Please find further product information on: **Shop.N-Ippin.com.**

## Attention

- Please use this product for the described purposes and application only. A disregarding may result in serious injuries.
- Please do not use this product if you suffer on high blood pressure, varicosity, thrombosis and similar clinical pictures, respectively turn to your doctor or podiatrist **before** using **Cinderella**.
- Necessarily keep this product out of the reach of children.
- The fabric consists of synthetic material. In case that skin rash or any dermatological reactions arises, please stop the appliance of **Cinderella** immediately and turn to your doctor if so.


## Care instructions

- Hand wash with 30° Celsius (86° F) only
- Please do not bleach, tumble and iron
- Please dry without direct sun radiation

## Removal

- This product consists of synthetic textiles. Please dispose considering your local waste regulation.

## Storing

- Please store dry and avoid direct sun radiation and high temperatures
- Please take care to sufficient ventilation 

## Material

- Nylon
- Polyurethane

European Shoe Size	UK Shoe Size	US Shoe Size	Sleeve Size	Foot Length
35.5 - 37.5	2.5 - 4	5 - 6.5	S	21 – 23 cm
38 - 40.5	4.5 - 6.5	7 - 9	M	23 – 25 cm
41 - 44	7 - 9	9.5 - 11.5	L	25 – 27 cm

Distributor:

Rhabanusstraße 38  
65375 Oestrich-Winkel  
Germany  
Tel: +49 (0) 67 23 – 8 08 11 10  
Fax: +49 (0) 67 23 – 8 08 11 12  
E-Mail: info@n-ippin.com  
Homepage: www.n-ippin.com  
Online-Shop: shop.n-ippin.com



**Daiya Industry Co., Ltd.**  
1253, Ofuku, Minami-ku, Okayama, Japan 701-0204

Batch No.: 306372 | Last update: September 2014



# How to put Cinderella on (Left foot for example)

1.



Please put on the big toe and little toe hood first and then pull up the sleeve to the instep

2.



Pull up the heel belt

In the event that wrinkles are arising, please make sure you even them before you slip into your shoes

Little toe hood

Big toe hood

Double layer structured inlet

Heel belt

